'PRODUCT NAME EYETEC® DIAMOND KNIFE CLEANING BLOCK
'PRODUCT INFORMATION AND INSTRUCTIONS FOR USE

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GB - INSTRUCTIONS FOR USE

Description

EYETEC® Diamond Knife Cleaning Blocks are designed to allow the surgeon to perform two specific operations.

- The removal of debris from the diamond blade following and during (if required) the surgical procedure.
- The polishing of the blade to remove any protein build up.

Intended Use

PVA sponge ENT products are designed to staunch blood loss after invasive surgery or traumatic injury in ENT surgery. The PVA sponge reacts quickly to absorb blood and body fluids. PVA is biocompatible and in some instances can be used for post-operative tissue support and to carry medication to the operative site (where specified in the Instructions for Use).

Diamond Knife Cleaning Blocks are designed to allow the surgeon to perform two specific operations.

- The removal of debris from the diamond blade following and during (if required) the surgical procedure.
- The polishing of the blade to remove any protein build up.

CAUTIONS

- This device is supplied STERILE and ready to use.
- The device is for SINGLE USE ONLY. Do NOT re-sterilise or re-use.
- Do not use if the packaging has been opened or damaged.
- This device is intended for use by trained medical persons possessing the requisite skill and experience to use the device in accordance with the prevailing standards of medical practice and in conjunction with the instructions for this device.
- The product comes into contact with bodily fluids, which can be contaminated. Care should be taken in the handling and disposal of the device after use to prevent contamination.
- CAUTION: US Federal law restricts this device to sale by or on the order of a physician.

**'PRODUCT NAME EYETEC® DIAMOND KNIFE CLEANING BLOCK****'PRODUCT INFORMATION AND INSTRUCTIONS FOR USE**

- Always hold the knife perpendicular to the cleaning block surface to prevent stressing the blade.

Incident Reporting

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established

Instructions

Thoroughly rinse the diamond knife with distilled water.

Moisten one end of the cleaning block with sterile water. (Note that the block will soften where moist).

Hold the moistened cleaning block between finger and thumb. Do NOT place on a hard surface.

Using a vertical up and down motion, pierce the wet end of the cleaning block with the knife blade several times.

Repeat the procedure on the dry half of the block to polish the diamond.

Sterilisation

This device is supplied sterile by Gamma Irradiation and are designated for SINGLE USE ONLY.

HAZARDS ASSOCIATED WITH THE RE-USE OF SINGLE USE ONLY DEVICES

1. Single use devices have not been validated for re-use.
If you re-use a device you may be held Legally Liable for the safe performance.
2. Cross-contamination and infection risks to patients. Including transmission of:
 - CJD & Variant CJD.
 - Prion Diseases.
 - Bacterial Endotoxins.
 - Hepatitis B & Hepatitis C.
 - Risks posed by HIV and AIDS
3. Device failure through material fatigue or degradation caused by initial use and design.
4. Patient injury from device failure and/or chemical burns from residue of decontamination agents absorbed into the materials

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