
INSTRUSAFT

XL TRANSPORT CONTAINER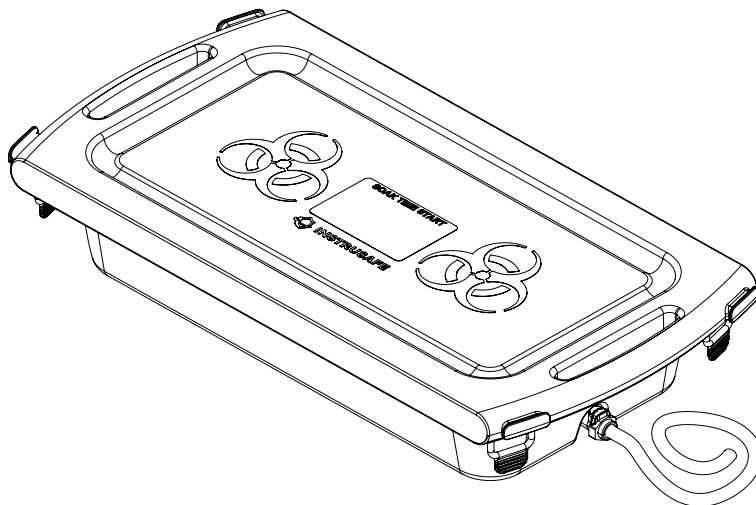

(EN)	XL TRANSPORT CONTAINER	INSTRUCTIONS FOR USE: English
(FR)	XL TRANSPORT CONTAINER	MODE D'EMPLOI: Français
(DE)	XL TRANSPORT CONTAINER	GEBRAUCHSANWEISUNG: Deutsch
(IT)	XL TRANSPORT CONTAINER	ISTRUZIONI PER L'USO: Italiano
(ES)	XL TRANSPORT CONTAINER	MODO DE EMPLEO: Español
(JA)	XL TRANSPORT CONTAINER	使用説明書: 日本語
(PT-BR)	XL TRANSPORT CONTAINER	INSTRUÇÕES DE UTILIZAÇÃO: Português
(NL)	XL TRANSPORT CONTAINER	GEBRUIKSAANWIJZING: Nederlands
(SV)	XL TRANSPORT CONTAINER	ANVÄNDARINSTRUKTIONER: Svenska
(DA)	XL TRANSPORT CONTAINER	BRUGSANVISNING: Dansk
(FI)	XL TRANSPORT CONTAINER	KÄYTTÖOHJEET: Suomi
(EL)	XL TRANSPORT CONTAINER	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ: Ελληνικά
(PL)	XL TRANSPORT CONTAINER	INSTRUKCJA UŻYTKOWANIA: Polski
(TR)	XL TRANSPORT CONTAINER	KULLANMA TALİMATLARI: Türkçe
(RU)	XL TRANSPORT CONTAINER	ИНСТРУКЦИЯ ПО ЭКСПЛУАТАЦИИ: Русский
(CS)	XL TRANSPORT CONTAINER	NÁVOD K POUŽITÍ: Český
(HU)	XL TRANSPORT CONTAINER	HASZNÁLATI ÚTMUTATÓ: Magyar
(SK)	XL TRANSPORT CONTAINER	POKyny NA POUŽITIE: Slovenčina
(NO)	XL TRANSPORT CONTAINER	BRUKSANVISNING: Norsk
(SL)	XL TRANSPORT CONTAINER	NAVODILA ZA UPORABO: Slovenski
(BG)	XL TRANSPORT CONTAINER	ИНСТРУКЦИИ ЗА УПОТРЕБА: Български
(RO)	XL TRANSPORT CONTAINER	INSTRUCȚIUNI DE UTILIZARE: Română
(ET)	XL TRANSPORT CONTAINER	KASUTUSJUHEND: Eesti
(LV)	XL TRANSPORT CONTAINER	LIETOŠANAS INSTRUKCIJA: Latviešu
(HR)	XL TRANSPORT CONTAINER	UPUTE ZA UPORABU: Hrvatski
(SR)	XL TRANSPORT CONTAINER	UPUTSTVO ZA UPOTREBU: Srpski
(LT)	XL TRANSPORT CONTAINER	NAUDOJIMO INSTRUKCIJA: lietuvių k.
(ZH-CN)	XL TRANSPORT CONTAINER	使用说明: 中文
(ZH)	XL TRANSPORT CONTAINER	請參閱使用說明書: 中文
(KO)	XL TRANSPORT CONTAINER	사용 지침: 한국어

TABLE OF CONTENTS	
LANGUAGE	PAGE
English	3
Français	4
Deutsch	5
Italiano	6
Español	7
日本語	8
Português	9
Nederlands	10
Svenska	11
Dansk	12
Suomi	13
Ελληνικά	14
Polski	15
Türkçe	16
Русский	17
Česky	18
Magyar	19
Slovenčina	20
Norsk	21
Slovenski	22
Български	23
Română	24
Eesti	25
Latviešu	26
Hrvatski	27
Srpski	28
Lietuvių k.	29
中文	30
中文	31
한국어	32

READ THIS SECTION BEFORE PLACING PRODUCT INTO SERVICE

INTENDED USE/ INDICATIONS FOR USE

InstruSafe® XL Transport Containers are intended to contain and protect reusable medical devices during transport from the operating room to the sterile processing department.

The InstruSafe® XL Transport Containers are used to collect used instruments and materials for transportation. The transport container can be used in a washer-disinfector and withstands repeated washing.

INTENDED USER

The InstruSafe® XL Transport Containers are intended to be used by healthcare professionals in the operating room and sterile processing departments for transportation of medical devices.

MATERIAL

The container is reusable and made of high temperature plastic. It is highly durable against heat and chemicals.

DESCRIPTION

The container consists of a lid and base, silicone gasket and latches. The gasket and latches can be removed to help facilitate cleaning and can be easily replaced. The container base holds the instruments and materials for transportation.

LID REMOVAL

Release the (4) silicone latches and remove the lid.

CONTAINER USE

1. Set container on stable surface
2. Add cleaning solution per the manufacturer's IFU to the container
3. Place instruments / materials in container for transportation
4. Place lid on container and engage (4) Silicone latches
5. Transport container
6. *For IN-PIC-20-D: To attach hose, press drain insert into coupling to open the valve
7. *For IN-PIC-20-D: To remove hose and close the valve, depress button and remove drain insert from coupling
8. *For IN-PIC-20-D: Drain should be flushed with water and air to remove any residue
9. Open lid slowly to remove instruments / materials
10. Process instruments / materials per the manufacturer's IFU

WARNINGS

1. **DO NOT** autoclave
2. **DO NOT** use as a sterile barrier
3. **DO NOT** drop as this may cause damage to the instruments / materials inside or damage the container resulting in possible leakage
4. **DO NOT** shake the container with liquids inside. This can cause damage to the contents or container leakage

Life Cycle: **DO NOT** use if container is cracked, broken or missing the silicone gasket or latches

5. Clean and inspect prior to use. Container may be reused until mechanical failure occurs
6. Be cautious of pinch points while lid is being opened and closed to avoid harm to fingers and hands
7. Be cautious of container weight with liquid added
8. **DO NOT** wash container with a hard brush. This can cause damage to the container
9. **DO NOT** use if package is damaged or unintentionally opened prior to use
10. Remove cover before automated washing
11. **DO NOT** stack if optional drain is installed
12. **DO NOT** stretch the silicone gasket


MANUAL CLEANING

1. Use a neutral (6.0 - 8.5) pH detergent
2. Remove all visible soil and contaminants using a soft brush
3. Immerse the container while cleaning to aid in the removal of contaminants and reduce splashing of detergent on personnel
4. Rinse thoroughly with clean water to remove detergent

AUTOMATED WASHER CLEANING


1. Container is safe for use in automated washers and can withstand up to 230°F

Gasket and latches can be replaced without the use of tools.
Gasket Replacement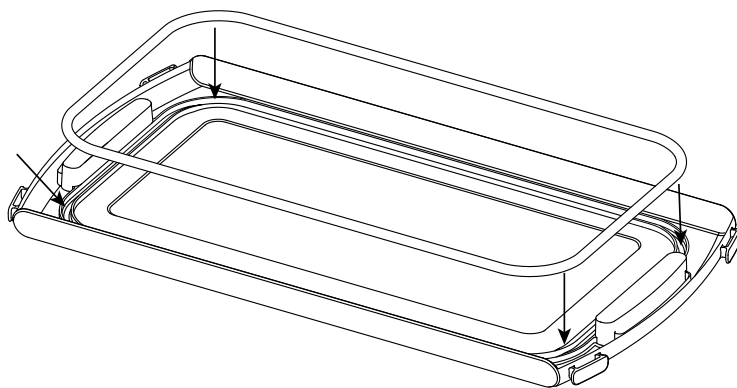

1. **DO NOT** stretch the silicone gasket
2. To insert, align gasket and press down

Latch Replacement

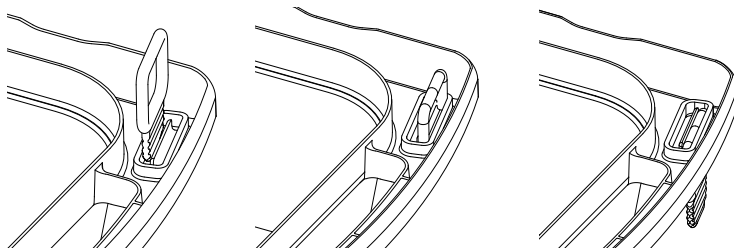1. Insert on top
2. Position latch over groove
3. Pull latch down into the groove

DISPOSAL

In the event the InstruSafe® XL Transport Containers do not pass inspection prior to use or have otherwise been deemed no longer fit for purpose, the devices shall be disposed of in line with local protocol. The method of disposal shall depend on the potential risks of cross-contamination and infection when the need for disposal is identified.

SERIOUS INCIDENT REPORTING

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the FDA/competent authority of the Member State in which the user and/or patient is established.

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InstruSafe® is a registered trademark of Summit Medical.
MDSS GmbH
Summit Medical LLC