


ClearView™ Scope Warmer v Thermo Bottle

Warming device	Start	Temperature of endoscope			End
	18°	25°	30°	35°	35°
Thermo bottle @ 37°	– ∇T = 19	– ∇T = 12	– ∇T = 7	– ∇T = 2	
ClearView™ @ 51°	– ∇T = 33	– ∇T = 26	– ∇T = 21	– ∇T = 16	

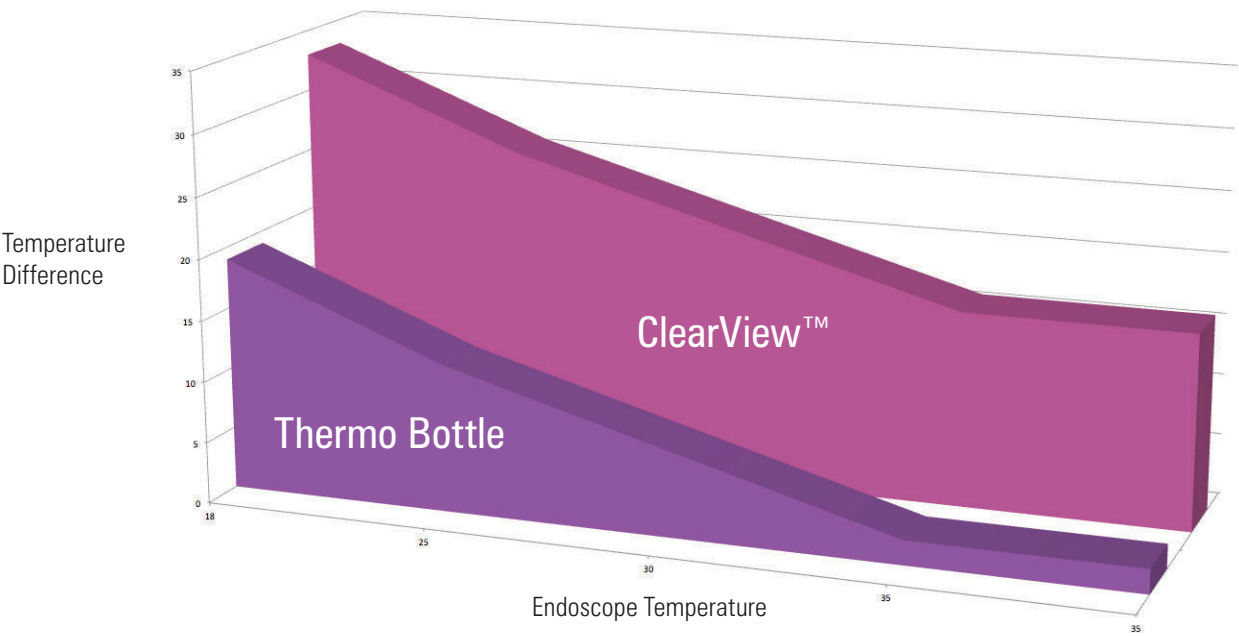

Benefits of ClearView™

- With a simple click of the built-in disc, warming will start immediately, no electricity or warm water is needed
- The ClearView™ Scope Warmer generates a significantly higher difference during the complete process of pre-warming the endoscope
- The ClearView™ exchanges 75% more energy at the start compared to using warm water
- The ClearView™ device warms full scope length to 37° within 90 seconds
- Within 5 minutes the ClearView™ Scope Warmer will reach a maximum temperature of 51°
- The heating reaction within the ClearView™ Scope Warmer will continue, staying above body temperature for 120 minutes
- For maximum benefit of warming the endoscope, it is advised to wrap the Scope Warmer in a sterile towel

ClearView™ Anti-Fog Scope Warmer PRODUCT CODE 31SW30



t +44 (0) 1792 797 910
e marketing@dtrmedical.com

 www.dtrmedical.com